

vatech Comprehensive shortcuts guide

General window navigation

Double click the title of each window	Single View window Return to the previous layout
Hover the mouse over each window	View focused
Press <Enter> key	Single View window Return to the previous layout
Press <Ctrl + Enter> key	Presentation Mode On/Off

3D View Window

Scroll the mouse wheel	Zooming
Click + Drag and Drop	Rotation
Right click + Drag and Drop	Panning
Double click	Smart Zoom
Press space bar on the keyboard	Show / Hide 3D Axis

2D View Window

Press <Ctrl> key + scroll the mouse wheel.	Zooming
<Shift> key + Right click + Drag and Drop	Panning
Scroll the mouse wheel	Change interval
Right click + Drag (left or right) + Drop	Adjust the width (brightness)
Right click + Drag (up or down) + Drop	Adjust the level (contrast)
Press space bar on the keyboard	Show / Hide 2D Axis
Double click	Move the center of axis to point clicked

Section Tab Window

Double click the title of Section window	Maximize Section window Return to the previous layout
Double click the Section View image	Maximize Section window (1X1 size) Return to the previous layout

